# MANUAL DE INSTRUCCIONES/ INSTRUCTION MANUAL/ MANUEL D'UTILISATION/ MANUALE D'ISTRUZIONI/ANWEISUNGEN: Columna, Capitel.

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Last revision date: April 14th 2021

## ESPAÑOL

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## ENGLISH

Please read carefully this direction to know the walker well before use.

Put the walker out of the box and place leg on the floor.

Unfold it by opening the lateral frames until you open the spring lock on the top.

Adjust the crutch to suitable height. Adjusting method: press the spring lock with finger to adjust to the suitable height, fix the middle screw nut tightly (if it has one), then it can be used.

Examine whether the arm pad and handle are loose or they have sting. Prevent the loose arm pad and handle from endangering you, and be careful of the sting hurting your palms.

Examine whether the anti-slip rubber tips of the crutch are tight and stable. If they are worn, change them immediately. Pay more attention when use on the slope.

### About walking:

First, put your walker about one step ahead of you, making sure the leg of your walker are level to the ground with both hands, grip the handles on the top of the walker for support and walk into it. Take small steps when you turn. To fold it, press the spring lock on the top and fold the lateral frames towards the front frame.

### Warning and precautions:

Make sure the walker is adjusted to the suitable height and the spring lock on four legs sticks out of the hole. Lock the Middle screw nut safely and tightly. Make sure the spring Lock on the top also sticks out from the hole on the top. Make sure lateral frames are completely extended before use. Check the rubber tip and handle in good working status, before usage. Avoid exposure of the product to sunshine. Keep and store it in cool and dry space. Avoid too much humidity. Don't grip the walker by the front bar and don't sit on it. Don't go up or down stairs using the walker. Weight capacity: 100kg. Remember this product should only be used on flat and smooth surfaces.

 **MOBICLINIC**  
Spanish Leading Company Since 1985

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD
SUNGO EUROPE B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.