MANUAL DE INSTRUCCIONES/INSTRUCTIONS MANUAL/MANUEL D'UTILISATION/ MANUALE D'ISTRUZIONI/ANWEISUNGEN: Pórtico/Compostela

ESPAÑOL

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ENGLISH

Read these instructions carefully in order to familiarize yourself with the walker before using it.

1. Put the walker out of the box and place its legs on the floor.
2. Unfold it by opening the lateral frames until the spring lock on the top is open.
3. Adjust the crutch to a suitable height. To do so, press the spring lock with finger to adjust to the suitable height, fix the middle screw nut tightly (if it has one), then it can be used.
4. Check that the arm pad and handle are tightly adjusted; be careful not to harm yourself if they are loose.
5. Examine whether the anti-slip rubber tips of the crutch are tight and stable. If they are worn, change them immediately. Pay special attention when using the product on slopes.

How to use the walker:

Place the walker on the floor about one step ahead of you. Make sure the legs are adjusted on the same height. Grip the handles with both hands and walk into it.

Take small steps when you turn. To fold it, press the spring lock on the top and fold the lateral frames towards the front frame.

Warning and precautions:

1. Make sure the walker is adjusted to your suitable height and that the spring lock on four legs sticks out of the hole.
2. Lock the middle screw nut safely and tightly. Make sure the spring lock on the top also sticks out from the hole on the top.
3. Make sure lateral frames are completely extended before use. Check that the rubber tip and handle are in good working status, before usage.
6. Avoid exposure of the product to sunshine. Keep and store it in a cool and dry space. Avoid too much humidity.
6. Don't grip the walker by the front bar and don't sit on it.
7. Don't go up or down stairs using the walker.

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