REFERENCE	
PEPE - ROLLATOR	P10007

The **Pepe Rollator** is an ideal walking aid that also allows the user to rest thanks to its seat with a folding backrest that can be adjusted according to the user's needs. It is designed to be used by elderly people or people with reduced mobility who need a walking aid.

FOLDING SYSTEM

Remove all packaging and examine all parts for signs of shipping damage. Do not use the rollator if it is damaged or any parts are missing.

When unfolding, pull the rear tubes backwards and push the attachment plates on both sides downwards until these lock in position. Place the basket and turn the seat downwards.

When folding, lift the seat upwards and pull the attachment plates upwards. Do not place your fingers between the folding mechanism. Push the rear tubes forwards against the front tubes and fold up the back rest by pulling these upwards.

HEIGHT ADJUSTMENT

1. Stand straight behind the rollator with your shoulder relaxed and arms slightly bent at 15°.
2. Adjust the height in one of the 6 settings so that the handles are at wrist height.
3. Set the correct height placing the handle height adjustment.
4. adjust the height of both handles and check that both have the same height.

BRAKE SYSTEM

Multifunctional brake system: The brake handles underneath the handles.

- To brake, pull the handle brake upwards toward the handle (1).

- To activate the parking brake, push the handle brakedownwards till you hear a locking sound of the brake. The wheels will be blocked until you pull the parking brakes upwards again.

WARNINGS

- This is a walking device for compensation and improvement respectively of restricted ability of walking.
- Ensure the Rollator is fully open and secured in the correct position before use.
- Test the Rollator for stability before use.
- Test the brake mechanism before use.
- The Rollator is a walking aid only and should not be used as a transportation device. Use only on even and solid surfaces, never on an incline or uneven ground.
- Do not stop, turn or reverse on a ramp or incline. Do not make sharp turns.
- Do not exceed the maximum user weight: 300 lb.
- Do not exceed the maximum basket load: 11 lb.
- Do not expose the product to temperatures above 104°F for a long time, otherwise the aluminium tubes cover heat, which could lead to burns.
- Ensure there are no objects blocking the wheels of the rollator.
- Only proceed moving with your rollator when you are standing comfortably and with both hands on the handles.
- Ensure the brakes or the rollator are always in parking-mode before using. When you are standing comfortable unlock the parking brakes.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.
- The Rollator should be inspected before each use to ensure it is free from damage and excessive wear. If you do find wear or damage to your Walker, DO NOT USE.
- Check that both wheels, tips and folding parts move freely and that there is no build-up of debris around the axle.
- Keep the Rollator clean by wiping with a damp cloth on regular basis. Do not use abrasive cleaning materials.
- If required, only use genuine parts supplied by PEPE MOBILITY or an approved stockist.
- PEPE MOBILITY warrants that the Rollator is free from defects in material and workmanship. The warranty shall remain in effect for 2 years from the date of original consumer purchase. The warranty does not extend to parts that are exposed to natural wear and tear that will require periodic replacement such as brakes, wheels, handgrips and bags. This warranty does not cover the following:
 - Replacement parts supplied by anyone other than PEPE MOBILITY or the stockist.
 - Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use.
 - PEPE MOBILITY is not liable for incidental or consequential injury or damage property.
 - Should it be necessary to return the Rollator, it is essential that it is adequately packed to protect it in transit, preferably in its original packaging, otherwise PEPE MOBILITY cannot accept any responsibility for transit damage to the unit.
 - This does not affect your statutory rights.
 - Please retain these instructions for future reference.
 - The product is suitable for reprocessing in line with manufacturer's recommendations.
 - Any serious incident related to the product should be reported to the manufacturer and the competent authority of the Member State in which they are user and patient set.

REFERENCIA	
PEPE - ROLLATOR	P10007

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