INSTRUCTION MANUAL/MANUAL DE INSTRUCCIONES/MANUEL D'UTILISATION/ MANUALE D'ISTRUZIONI/ANWEISUNGEN: Rollator Future

ENGLISH

Please read carefully this direction to know your new rollator well before use. Future model has a weight capacity of 136kg.

The followings are steps of using rollators:

1. Place both feet side-by-side in the middle of the rollator frame.
2. Grasp the hand grips on either side of the rollator.
3. Push the rollator forward to a comfortable distance.
4. Step forward toward the inside of the rollator frame with your weaker leg.
5. Step forward with your stronger leg while supporting yourself with the hand grips. Both feet should be side-by-side between the wheels of the rollator. Continue walking by repeating Step 4 and Step 5.

Tips and Warnings

Keep a part of your body, like your toes, within the rollator frame at all times. When standing from a sitting position, push up from the seat instead of pulling up on the rollator handles. When turning, take small steps and keep your hips parallel to the front bar of the rollator. Avoid twisting your body away from the rollator. Ensure all wheels of the rollator are on level ground before walking. Don't hunch forward over the rollator. Don't use the rollator on an escalator or stairs.

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FOSHAN SUNCARE MEDICAL PRODUCTS CO., LTD.
SUNGO Europe B.V.